


IntelViewer Install and Basic Configuration Guide

1. **Browse** to <https://pacs.sydneyimaging.com.au/IntelViewer-tracks/evaluation/> via your web browser
 - a. Microsoft Internet Explorer
 - b. Microsoft Edge
 - c. Mozilla Firefox
 - d. Apple Safari
 - e. Google Chrome
 - f. Any other internet browser

Based on the operating system and architecture of your operating system, you will need to **select** which installer to download.

In this example, I am running Windows 10 64-bit edition and will need to select the 64-bit Windows installer and **click** Download IntelViewer under that menu (example below).

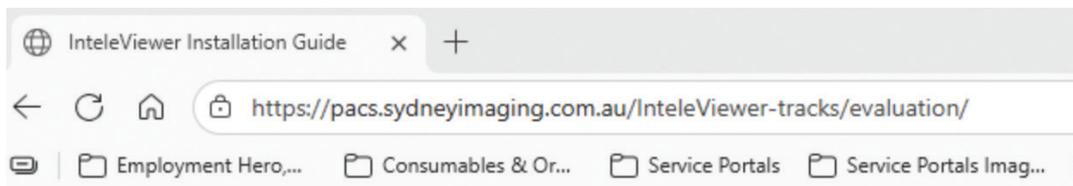

64-bit Windows Installer


[Download IntelViewer MSI](#)

[Additional Installation Options](#)

Mac OS X Installer


[Download IntelViewer](#)

2. Wait as IntelViewer downloads

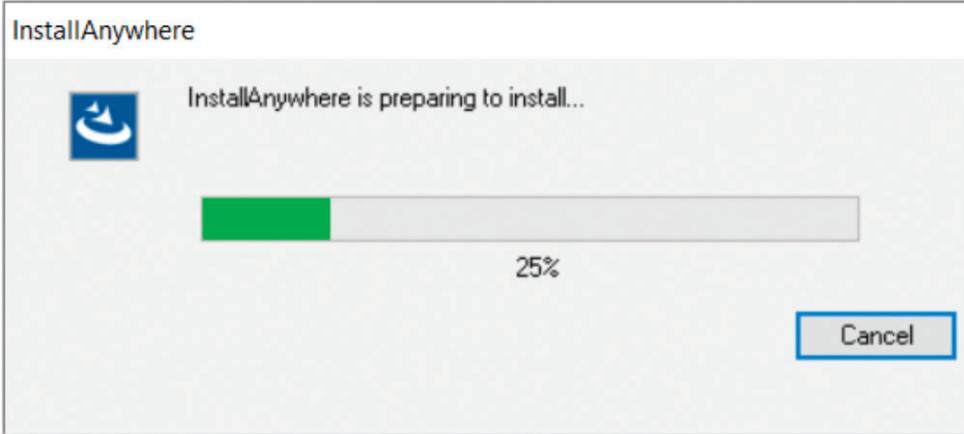


3. Once loading is complete, please go to the folder where it downloaded (generally it is in your Downloads folder) and run the installer as an administrator (if possible).


Hint: You can access this via the Quick access menu in your file explorer window on Windows 8/10.

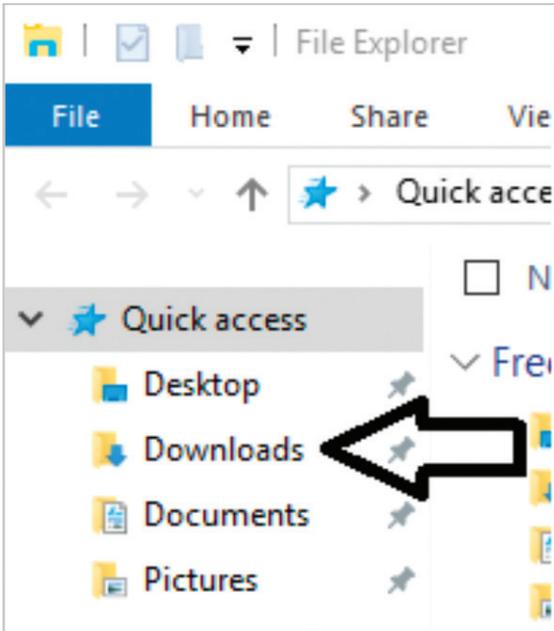

4. **Double click** the downloaded file and it will open.

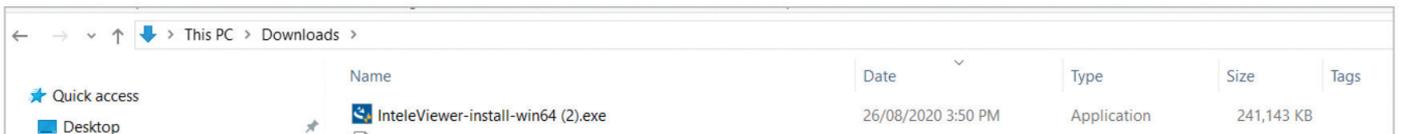


5. Please wait while the installer initialises

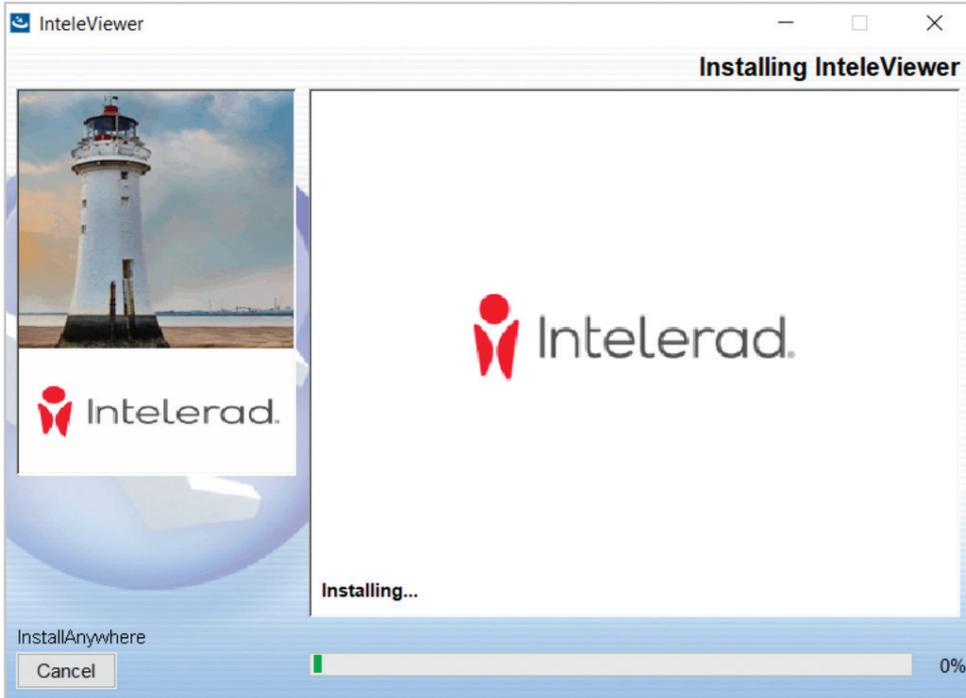


6. IntelViewer's license and terms & conditions of use will open. To proceed with the installation **select** "I accept the terms of the License Agreement" and then **click** the **Next** button.




7. You will also need to agree to Oracle's Java licensing. To proceed with the installation please **select** "I accept the terms of the License Agreement" then **click** the **Next** button.

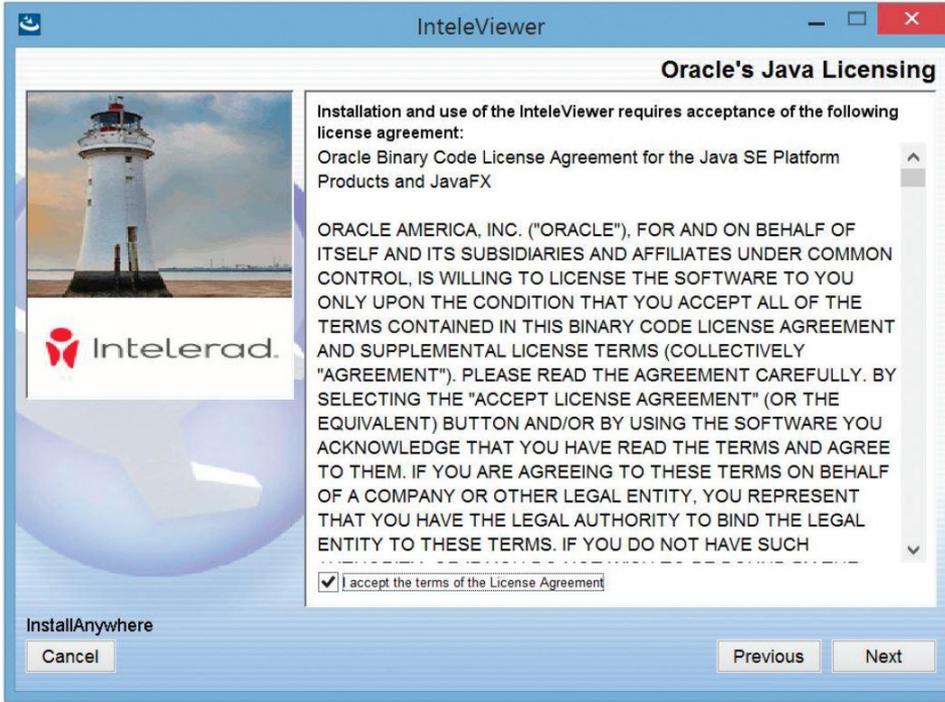


8. The installation will ask where you would like to install IntelViewer, in this example we will leave the installation folder as default. It is only in rare cases where you may be required to install it to another folder. Please **click** the **Next** button to proceed.

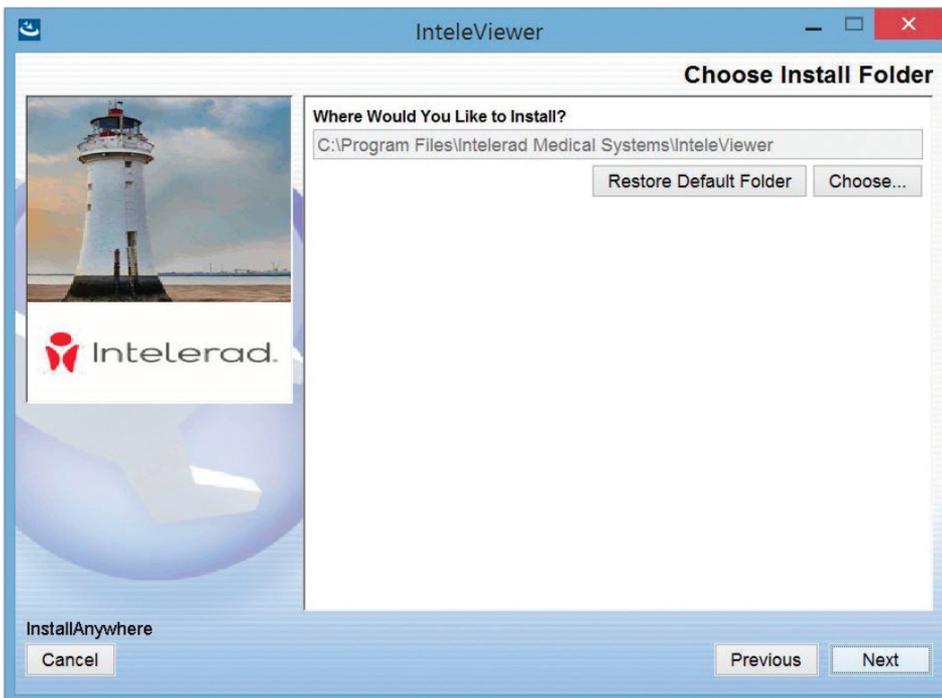

9. The next window will ask if you want all users to be able to upgrade IntelViewer. This is solely up to the end-user. In this example we will allow all users, as this guide is installing the stable track which means the upgrade process will only install the latest stable version. Please **click** Next

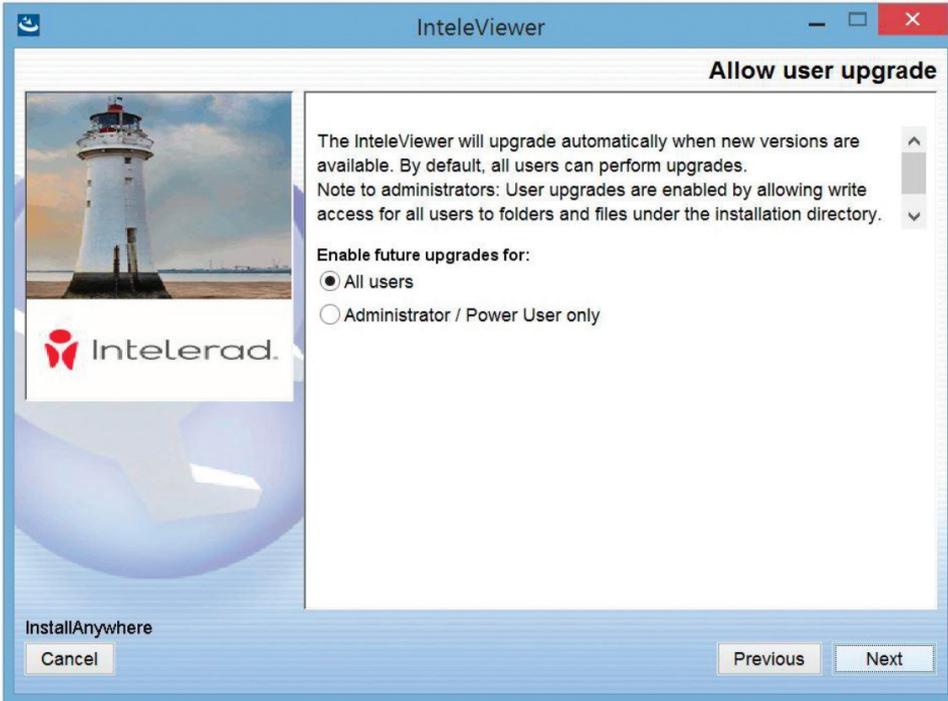


10. Please leave this disabled as this is not required, please **click** next

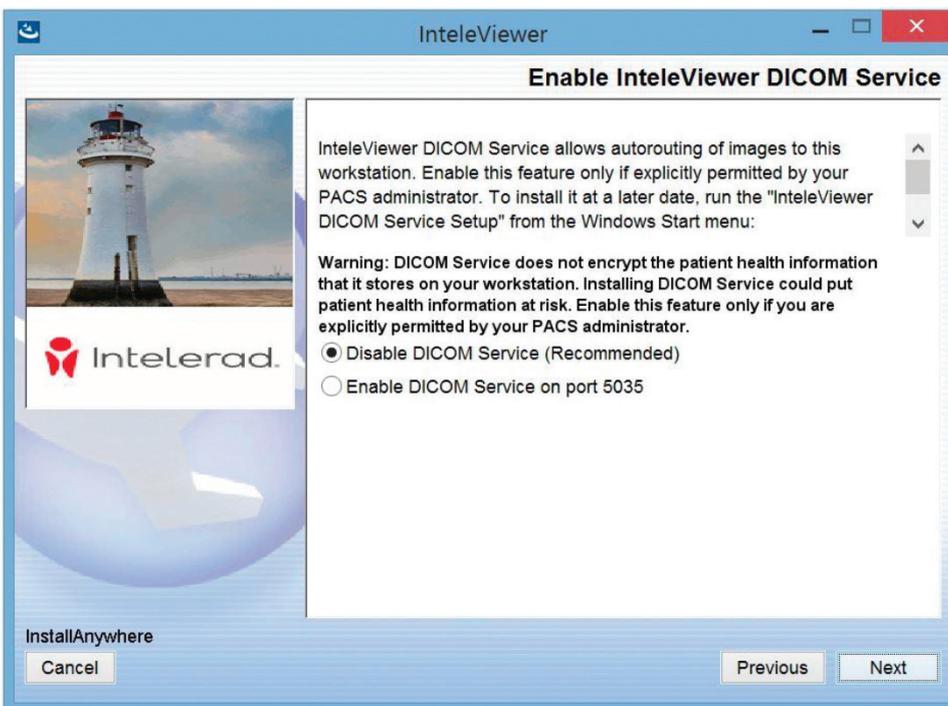

11. Please wait as IntelViewer installs




12. Once the installation has completed the installation window will disappear.
13. The IntelViewer installation will create the following shortcut:

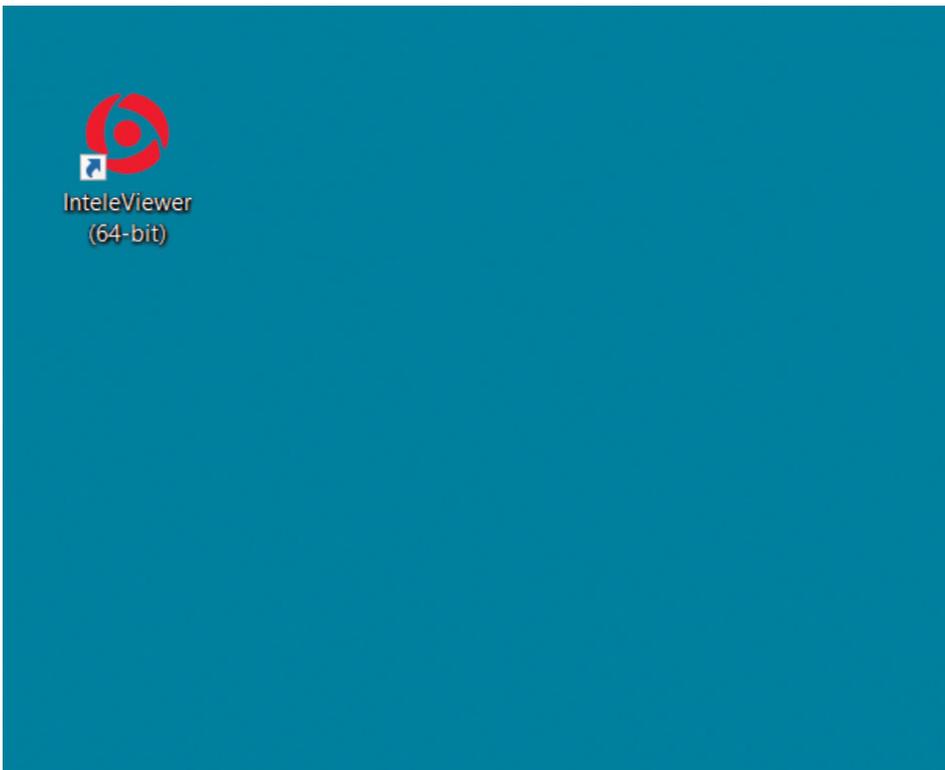


14. When you open IntelViewer for the first time, a prompt will appear asking which server you want to connect to. Please **click** the button Enter a New Server.



15. The new bookmark window will open.

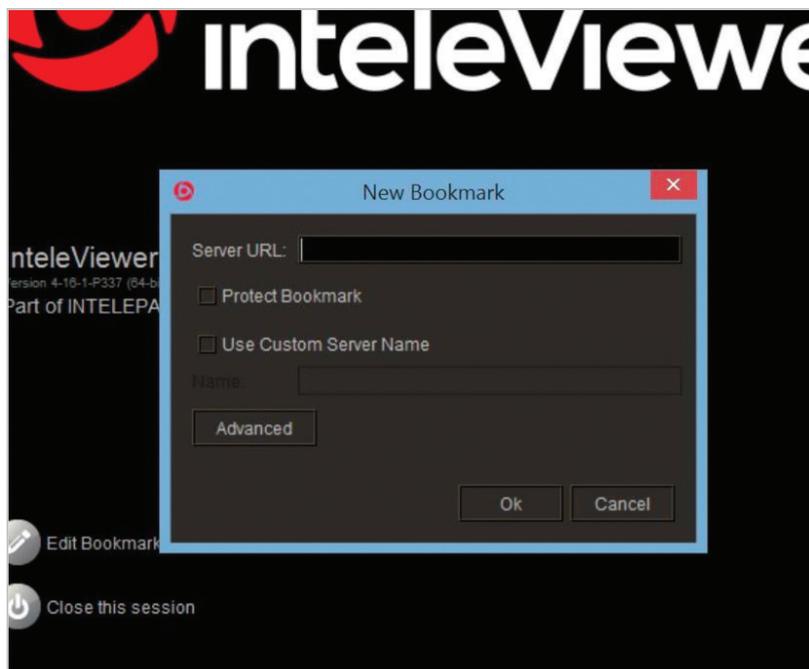

16. Type the complete URL on the next line in the Server URL text box and **click** the OK button: <https://pacs.sydneyimaging.com.au/>

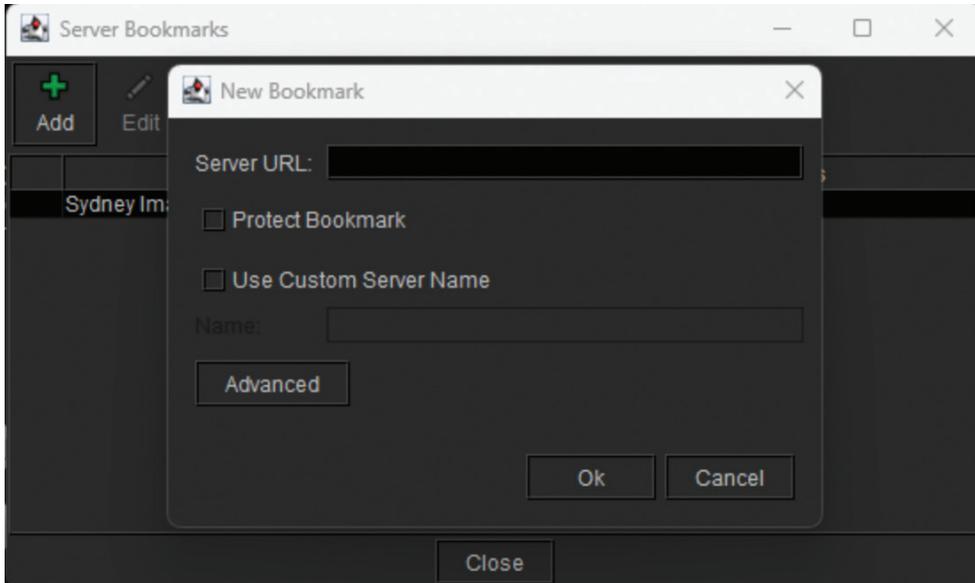

17. Once the address is verified, the bookmark will display in the IntelViewer window.
Please note: The bookmark will not be able to be added without internet access.



18. Please **type** in your username and password and **press ENTER** on your keyboard or **click** the right pointing arrow button in the IntelViewer window



For further assistance, please contact our Business Development Team on (02) 9044 3200.